
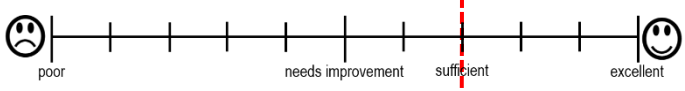

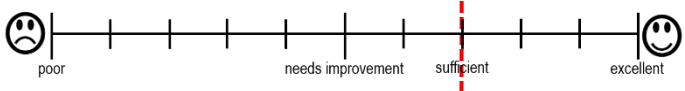



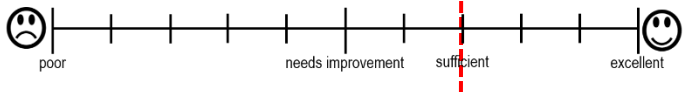


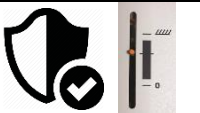
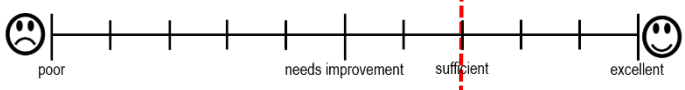

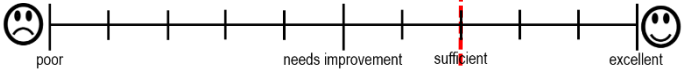
Lokomat® User Requirements Checklist

In order to ensure safe, effective and efficient use of the Lokomat, the clinicians that are certified as users must have a certain level of practical knowhow before using the device on their own. This checklist serves as a support for you as a Trainer when you educate new users on the Lokomat to check if they are ready to work with the Lokomat. Before certifying a clinician as user, he/she should fulfil the criteria below:


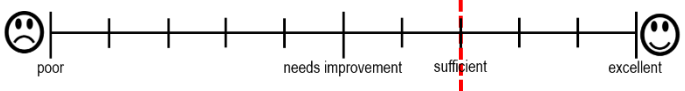

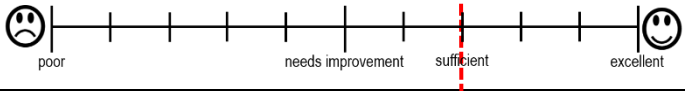

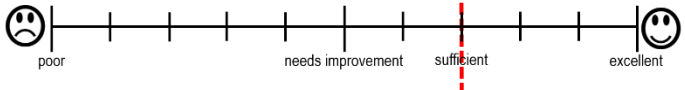
1. Prepare the Patient and the Lokomat and Put the Patient in the Lokomat

		Description	Name of the Candidate: _____
1		Candidate knows indications and contraindications	
2		Candidate knows the criteria for the selection of the orthopedic material and applies it correctly	
3		Candidate knows the criteria for proper alignment and fixation of hip, knee and ankle and can apply them	
4		Candidate knows the criteria for proper alignment of sagittal plane and frontal plane and can apply them	

2. Training procedure: Safe and Physiological Walk

1		Candidate can perform the adjustments to ensure a safe walk in the dynamic range (correct/avoid foot-drop and unsafe knee flexion/extension)	
2		Candidate knows the adjustments to achieve a physiological walk	

3. General Aspects

1		Candidate can handle the patient in the device safely and knows how to handle an emergency situation	
2		Candidate can handle the software adjustments : speed, BWS, GF, ROM, offset, synchronization	
3		Candidate knows different options how to challenge a patient	

To become a User, all ratings should at least be 'sufficient'

Candidate passed the User Education:

No

Yes

Name and Signature of Lokomat Trainer

Name and Signature of Lokomat User

Name	Signature

Name	Signature

Date/Place: _____ / _____